

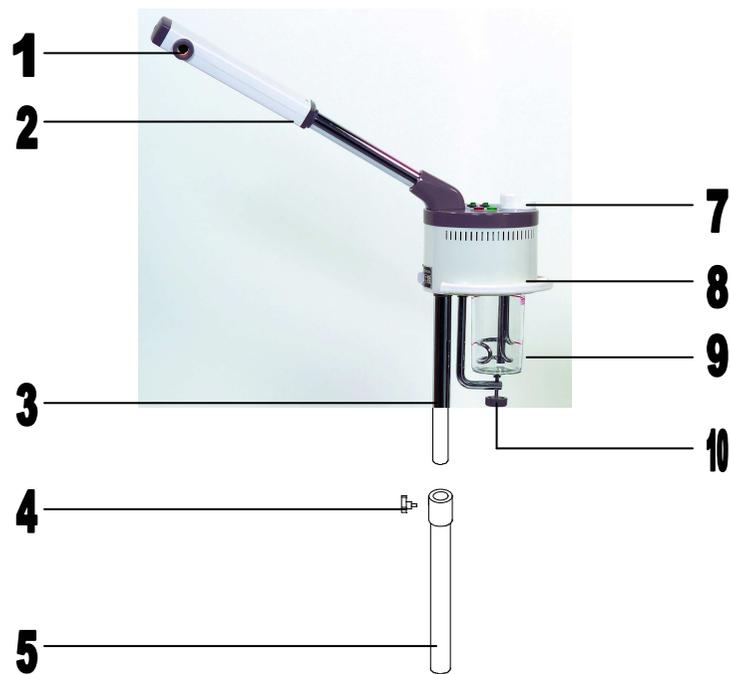
Mod. 310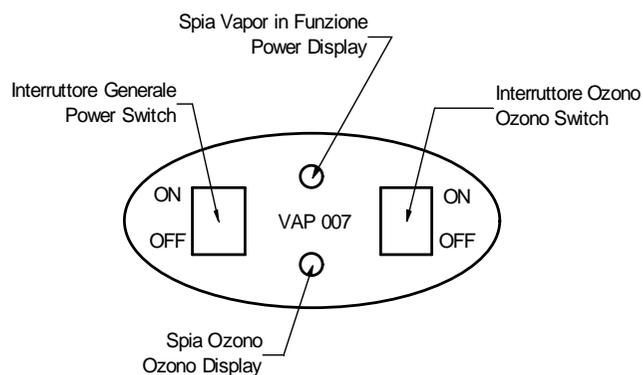

- 1) A in B
- 2) B in C
- 3) D in C

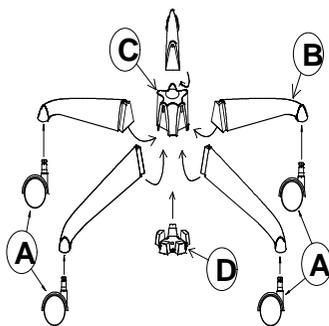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PHASES OF INSTALLATION:

1. To insert the inferior part of the pedestal in you cover base
2. To screw the inferior wall of the pedestal with the bolt
3. To insert the advanced part of the pedestal in the vaporizzatore
4. To mount all and to screw the knob (in equipment) in the inferior part of the pedestal

Importato da / Imported by
B.M.P. s.r.l. - 42019 SCANDIANO (RE) Italy
 Tel. +39 0522 856064 - Fax +39 0522 983499
 e-mail: info@bmpsrl.it - www.bmpsrl.it

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FUNCTIONS:

Ionize vapor generator, can facilitate the circulation of the blood and the metabolic processes, rinse the skin and enrich the hydration of the cells. The skin will be improved and more delicate.

HOW TO USE:

- A To fill up the appropriate cup with distilled water, having care not to exceed the indicated limit.
- B Connect it to electricity (220V) and push down the start switch. After approximately 6/8 min. the vapor will begin to come out from the spray nozzle. Press the ultraviolet switch and the Vapor will begin to radiating out the ionized vapor.
- C In the case that some water drops falls under the heat conductor, automatically the machine will extinguish. If a frequent use is previewed, it is recommended to detach always from the electricity from one sitting to the other, for being able to recharge the machine; empty the cup accurately, wash it and still fill up it .
- D The position of an corrected use of the machine allows them to blow from the chin up, until covering the entire face. The duration of the sitting depends on the type of skin to deal.

PRECAUTIONS:

- A During the installation and the regulation of the vertical tube, to hold the high part of the tube firm while the just height was regulated with care in the movements.
- B Don't put inside of the cup any type of chemical products and additives or otherwise could to come out water squirts.
- C When the vapor is used, always attend until that the vapor not decoy regularly and in continuous way; only at this point to aim the spray nozzle against to the human body.
- D After every use, empty the appropriate cup and wash it with care. In the case of calcium oxide remain in the heat conductor, remove them with one soft brush of metal or dip them for one night in a vinegar solution. Not use detergents to wash the cup.